





Displaying Live Images While Using the AFHCANcart

During Vidyo Session:

1. Join the Vidyo session.
2. Turn on imaging device attached to the cart (JedMed Otoscope, Welch Allyn Otoscope, Versa 4 Exam Camera, AMD 2500 Exam Camera, regular camera, dental camera)
3. Attach appropriate lens and/or specula to the device
4. Select Settings on Vidyo tool bar
5. Select your imaging device and click APPLY. This changes the camera being used from your webcam to your device.

NOTE: It is not ALWAYS clear which device to select. You may have to use trial and error to find the correct device. Webcam usually has "webcam" in the name, devices you plug in to a USB port should have "USB" in the name, etc.

6. Close or minimize settings window.
7. When you no longer need to share the image, select Settings on the Vidyo toolbar.
8. To go back to your vidyo session, select the Logitech or other web camera that was previously in use in your cart.

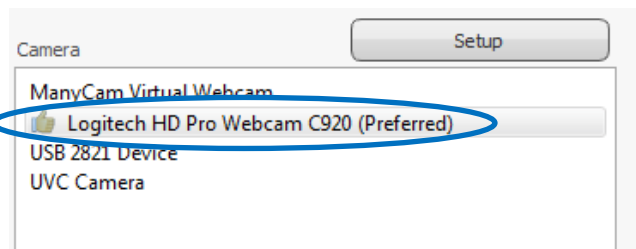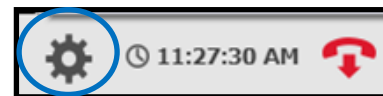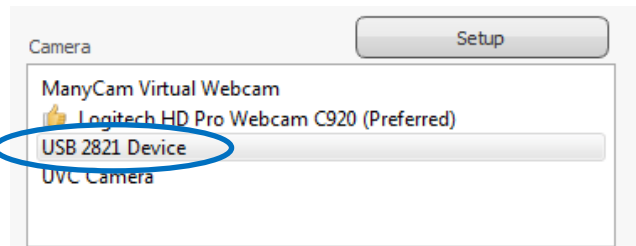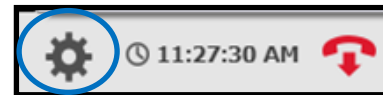




For the following Handy Guides, go to: <http://www.afhcan.org/support.aspx>

- HG-0016 How to Capture Images Using the Jed Med Otoscope
- HG-0021 Welch Allyn Otoscope
- HG-0244 Versa 4 Multi-Purpose Exam Camera
- HG-0250 AMD-2500 Exam Camera
- HG-0277 Using the Evolution Dental Camera
- HG-0241 Iris Dental

Sharing your AFHCANcart case or your monitor screen on Vidyo

During a Vidyo Session:

1. From within Vidyo, hover over the bottom of the screen until the Vidyo tool bar appears
2. Click on the Share button
3. If you are already on AFHCANcart with a case open: select the "AFHCAN Telemedicine" option from the Share button. You can continue to work on your case and all that you do within the case will be shared live via Vidyo.
4. If you are on the cart without a case open: select the "Monitor" option. Whatever you see on your monitor will be shared live via Vidyo.
5. The Toggle Share icon will change to green when you are actively sharing.
6. To stop sharing, hover over the bottom of the screen and click on the Share button
7. Select Stop Sharing

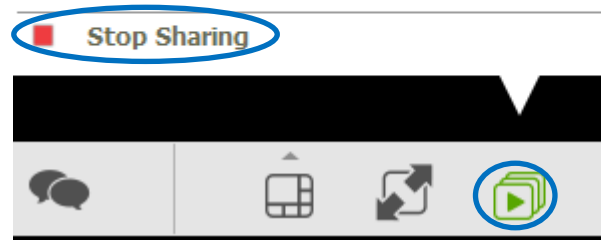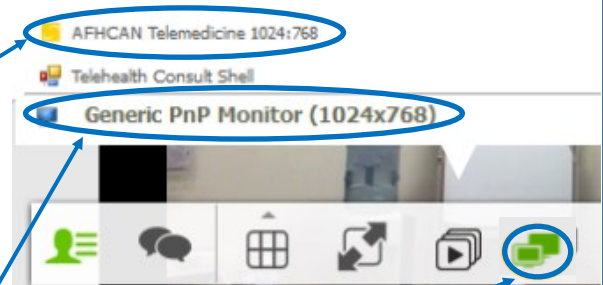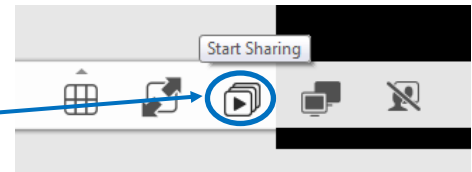

NOTE: Every user can toggle between all shared items within a session by clicking on them in the Share button list. They can also stop viewing by going to the Share button and selecting Stop Viewing Share.

